



# ONLY YOU CAN SEE THE LIGHT


Featuring the SKEDCO Combat Applications Laryngoscope Kit™  
(SK-CAL-CB/SK-CAL01-CB)

Perform Tactical Field Care and Combat CASEVAC Skills at Night, Under NVGs

Endotracheal Intubation or Peripheral IV Insertion Under NVGs...No Problem

Equipped with the "Combat Proven" Surefire 6P Flashlight, IR Filter and the Revolutionary New Fiber Optic Laryngoscope Adapter by SKEDCO...  
(TACTICAL LARYNGOSCOPE™)

## YOU CAN OWN THE NIGHT

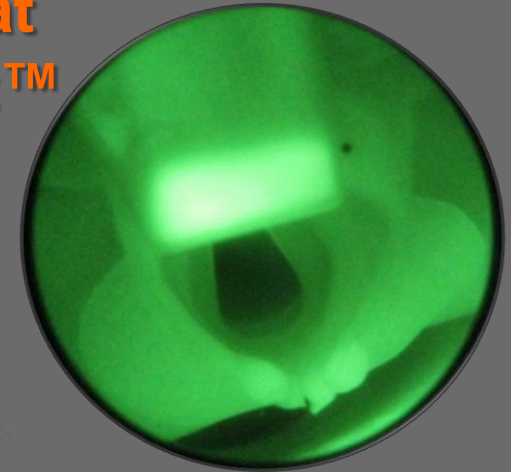

**NOTE:** Use incandescent lights ONLY; LED lights are NOT compatible with NVGs and IR filters, and WILL NOT function.

# Introduction

The **SKEDCO Combat Applications Laryngoscope (SCAL) Kit™** utilizes an extremely durable, light-weight, revolutionary adapter. The adaptor quickly converts a standard SUREFIRE 6P Flashlight into a “**Tactical Laryngoscope**” Handle; ready to accept a fiber optic laryngoscope blade (Miller/Mac). Tactical Intubation at night with NVGs; just insert the kit’s IR filter into the adapter, connect your blade and your “Tactical Laryngoscope™” is NVG compatible. Is starting an IV at night under NVGs a challenge? The Combat Applications Laryngoscope aids in this task by highlighting veins in the hands; making IV insertion possible.

## A Word of Caution

While the SKEDCO Combat Applications Laryngoscope Adapter is specifically designed for use with the original SUREFIRE 6P Incandescent Flashlight (1.25” Bezel Ring). This adapter also works with the following Surefire flashlights: 9P, G2, and G3. While the adapter will work with LED versions of the above lights, only incandescent lights can be used to intubate with NVGs and IR blackout filters. **LED lights are NOT compatible with NVGs and IR filters, and WILL NOT function.**

It is strongly recommended that following each live patient use, the laryngoscope adapter and blades be disposed of IAW unit biohazard protocols. Under normal use, the IR filter should be protected from biohazard exposure and may be used again, but in case of exposure, dispose of properly. Follow manufacture guidelines to decontaminate the SUREFIRE 6P Flashlight.



## SK-CAL-CB and SK-CAL01-CB\* Sub-Components

**\*W/O SUREFIRE 6P**

- **SUREFIRE 6P Flashlight\***
- **SKEDCO Laryngoscope Adapter**
- **# 4 Miller Blade (Disposable)**
- **# 4 Mac Blade (Disposable)**
- **IR Filter**
- **SKEDCO Custom Designed Carrier**