



CELOX™ MEDICAL

MADE TO SAVE LIVES

**WHEN IT'S LIFE
OR DEATH, EVERY
SECOND COUNTS
THINK CELOX™**

PRODUCT BROCHURE

THE CHALLENGE: SAVING LIVES FROM POST-TRAUMA BLEEDING

Catastrophic bleeding is the leading cause of preventable death, both in civilian life^{2,3} and on the battlefield.⁴




ACCOUNTS FOR
1 IN 10
MORTALITIES⁵




SIGNIFICANT
SOCIAL
AND ECONOMIC
BURDEN⁶




TIMELY
ACCESS TO
CARE



DEATH TOLL
INCREASING
YEAR-ON-YEAR⁵

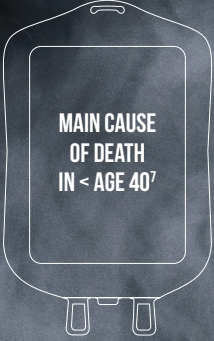

MAIN CAUSE
OF DEATH
IN < AGE 40⁷

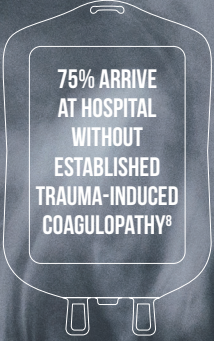

75% ARRIVE
AT HOSPITAL
WITHOUT
ESTABLISHED
TRAUMA-INDUCED
COAGULOPATHY⁸

THE SOLUTION: **CELOX STOPS BLEEDING FAST AND HELPS SAVE LIVES**

Celox trauma products have helped save the lives of soldiers and civilians since 2006.

- + Celox haemostats use a high-quality chitosan formulation, scientifically developed as a fast-acting, safe and highly effective hemostat
- + Celox works independently of the body's classic coagulation pathway and does not generate heat⁹
- + Celox haemostats can rapidly promote clot formation, through absorption and dehydration forming a gel-like plug
- + Celox granules work – even in challenging cases, including hypothermia and anticoagulation medications, e.g. heparin, warfarin^{9,10}



CELOX RAPID

The most recent addition to the Celox product range is Celox Rapid, our fastest-acting, high-density haemostatic gauze.

- + Celox Rapid contains our latest haemostatic technology with patented Chito-R augmented chitosan, giving fast emergency treatment in critical life-threatening situations
- + Celox Rapid is the UK Ministry of Defence's product of choice¹¹
- + Celox Rapid reduces blood loss by 60% by speeding up packing time and reducing compression time to one minute, or until bleeding stops¹²⁻¹⁴
- + Faster treatment time may improve casualty survival and aid treatment of multiple casualties

PRODUCT BENEFITS:

- + Works with one-minute compression, or until bleeding stops¹⁴
- + Rapid action reduces blood loss¹
- + Stops hypothermic bleeding¹⁶
- + Faster packing time than another leading brand¹⁴



IDEAL USE

LIFE-THREATENING BLEEDING INJURIES + CRITICAL SITUATIONS WHERE COMPRESSION TIME IS LIMITED + EMERGENCY, PRE-HOSPITAL CARE

HOW CELOX RAPID WORKS

1

Celox Rapid contains Chito-R,TM a high-quality chitosan derivative, with the addition of a pharmaceutical-grade bioadhesive

2

Chito-R is bonded to a high-density gauze which allows faster packing and greater adherence to surrounding tissue compared to another popular brand.¹⁵ This creates a secure gel plug for maintained haemostasis

3

Chito-R activated chitosan granules stick to wet tissue in the wound, improving the dressing's tamponade effect and controlling blood loss¹⁴



PRESENTATION: 1.5m long x 7.6cm wide 'Z-fold' gauze



THICKNESS: 0.68mm



ABSORBENCY: >500%

PRODUCT CODE: FG08839021

CELOX GAUZE

Celox Gauze is recommended by the US Committee on Tactical Combat Casualty Care (CoTCCC) and is the haemostat of choice for multiple NATO forces and first responder units around the world.

- + Celox Gauze is a highly effective dressing for haemostasis and is designed to stop bleeding from arterial injuries, gunshot wounds, road traffic accidents and other life-threatening bleeding injuries
- + Statistically significant improvement in survival, especially in more severe injuries¹
- + Reduced blood loss or requirement for fluid resuscitation with Celox Gauze versus another popular brand¹²

PRODUCT BENEFITS:

- + Highest observed survival rates in US Department of Defense testing¹⁷
- + Works independently of the body's clotting mechanism¹⁰
- + Stops hypothermic bleeding¹⁶
- + Works with oral anti-coagulant agent e.g. warfarin¹¹
- + Stops life-threatening emergency bleeding



IDEAL USE

LIFE-CRITICAL SITUATIONS, INCLUDING ARTERIAL INJURIES, GUNSHOT WOUNDS, ROAD TRAFFIC ACCIDENTS AND OTHER EMERGENCY BLEEDING INJURIES

HOW CELOX GAUZE WORKS

1

Celox Gauze was developed to provide an improved method of delivering the proprietary granules and comprises macroscopic, high surface area, granular flakes, for maximum effectiveness

2

Celox Gauze absorbs blood fluid, swells and forms a gel-like plug that covers the wound and stops the bleeding

3

Independent comparative tests in the form of a meta-analysis reported a statistically significant improvement in survival with Celox Gauze, compared to alternative agents ($P < 0.5$)¹



PRESENTATION: 1.5m long x 7.6cm wide 'Z-fold' gauze or 3m x 7.6cm roll gauze



THICKNESS: 0.68mm

ABSORBENCY: >500%


PRODUCT CODE: Celox Gauze FG08839031 (Z-fold, 5ft), FG08834011 (rolled, 10ft)

CELOX GRANULES

- + The original Celox product – proven to save lives and reliably stop arterial bleeding
- + Celox Granules mould to the shape of the wound, even in complex shaped injuries
- + In an independent trial by the US Navy, when compared to another popular agent and standard gauze, Celox Granules was the only product to give 100% survival and a robust gel-like clot, with no re-bleeding¹⁷

PRODUCT BENEFITS:

- + Significantly and quickly reduces blood loss
- + Treats irregularly shaped wounds and multiple injuries
- + Easy to use and remove¹¹

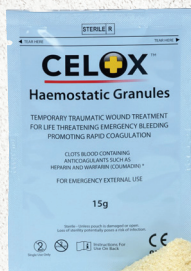

IDEAL USE

EMERGENCY, PRE-HOSPITAL CARE + ARTERIAL AND VENOUS BLEEDING
+ LIFE-THREATENING BLEEDING + COMPLEX WOUNDS

PRESENTATION: 15g

PRODUCT CODE: Celox Granules FG08830181

CELOX-A (APPLICATOR)

- + Celox-A is a high-speed applicator delivery system pre-packed with Celox Granules and designed to get through a small entry wound, directly to the source of bleeding, in just a few seconds.
- + Celox-A has excellent results in independent testing, demonstrating higher penetrating injury survival, compared with another popular agent and standard gauze¹⁷
- + Celox-A Granules are easily removed¹⁰

PRODUCT BENEFITS:



LIFE-THREATENING, EMERGENCY BLEEDING, PRE-HOSPITAL + SMALL ENTRY WOUNDS E.G. KNIFE OR BULLET + PENETRATING WOUNDS + SHRAPNEL WOUNDS FROM BLASTS
+ APPLYING THROUGH STRONG BLOOD FLOWS

PRESENTATION: 6g granules in a pre-filled applicator.

PRODUCT CODE: FG08832021


FOXSEAL™

+ A latex-free occlusive dressing for the treatment of open chest wounds and the prevention of pneumothorax


+ Unlike other dressings – which, due to their size, require folding, leading to a permanent crease that compromises a true seal – Foxseal is compact and neatly fits into pouches to avoid creasing

Provides dependable adhesion in the most challenging situations.¹⁸

Adhesion at 55°C

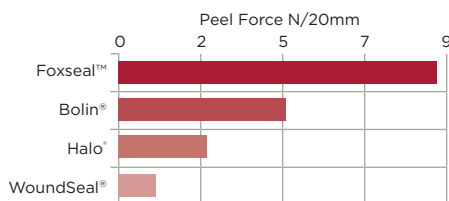

Adhesion at -18°C

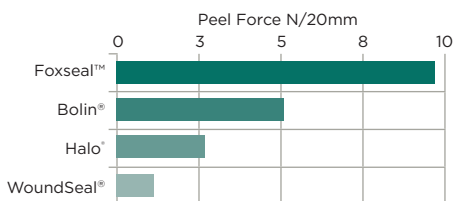

PRODUCT BENEFITS:

+ Two seals per pack allow entry and exit wounds to be treated

+ Simple, valve-free option allows sustained patient monitoring

+ Chest can be vented with the easily-gripped tab and adhesive re-seals



IDEAL USE

POST-TRAUMATIC BLEEDING ON OPEN CHEST WOUNDS + PRE-HOSPITAL EMERGENCY SITUATIONS

② **PRESENTATION:** 13.0 x 13.3cm/5.12" x 5.23" two single-use chest seals in a sterile pack

PRODUCT CODE: FG08814441

ORDERING CELOX PRODUCTS

Celox is sold through a network of distributors. To find out who represents Celox in your area, or to become a distributor, please contact **celoxenquiries@medtrade.co.uk** detailing what country you are in, and with details about your intended use.

Further information

To request a Material Safety Data Sheet (MSDS), or for any other information on the Celox product range, please contact **celoxenquiries@medtrade.co.uk**

REFERENCES

- 1 Winstanley M, *et al. J R Army Med Corps* 2018;0:1-5
- 2 World Health Organization. Cause-specific mortality and morbidity. 2009 http://www.who.int/whosis/whostat/EN_WHS09_Table2.pdf. Accessed 08/10/18
- 3 Cothren CC, *et al. World J Surg.* 2007;31(7):1507-11
- 4 Sharrock AE, *et al. Injury* 2018. Doi: 10.1016/j.injury.2018.06.037
- 5 Murray CJ, Lopez AD. *Lancet.* 1997;349(9064):1498-504
- 6 Campbell HE, *et al. Crit Care.* 2015;19(1):276
- 7 Kauvar DS *et al. J Trauma* 2006; 60:S3-S11.
- 8 MacLeod JB, *J Trauma.* 2003;55:39-44.)
- 9 Millner RWJ, *et al. Eur J Cardiothorac Surg* 2011; 39(6):952-954
- 10 Khoshmohabat H, *et al. Trauma* 2016;21(1):e26023
- 11 Celox. UK MoD Selects Celox Rapid Haemostatic Gauze for all Branches of the UK Military. *Cision PR Newswire*, 31 Jan, 2018, 15:48 GMT. Retrieved: 18th Oct. 2018 - link: <https://prn.to/2J51zsl>
- 12 Kunio N *et al. Am J Surg* 2013; 205: 505-510
- 13 Hoggarth A, *et al.* Mechanism of action of a rapid-acting gauze hemostat. Poster presentation at ATACCC 2011, Fort Lauderdale, FL
- 14 Hoggarth A, *et al.* Reduced application time with a rapid packing gauze haemostat. Poster presentation at ATACCC, FL, August 2011
- 15 Kheirabadi BS, *et al. J Trauma* 2009;66(2):316-26
- 16 Eastridge B, *et al. J Trauma* 2012; 73(6): S431-S437
- 17 Littlejohn LF, *et al. Acad Emer Med.* 2011; 18: 340-350
- 18 Observed in laboratory testing. Medtrade data on file
- 19 Rall J, *et al. J Trauma* 2013; 75 5150-5156



CELOX™ MEDICAL

MADE TO SAVE LIVES

**SPEED CAN KILL,
BUT IT CAN ALSO SAVE LIVES
THINK CELOX™**

MEDTRADE PRODUCTS LTD.

Electra House, Crewe Business Park, Crewe, CW1 6GL, UK

T: +44 (0)1270 500019 celoxenquiries@medtrade.co.uk www.celoxmedical.com

Copyright © 2021 Medtrade Products Ltd. CELOX™ is a trademark of Medtrade Products Ltd. All rights reserved.

MT-18-170-CE