Electronic Instruction For Use

* Please note that the images in this online instruction for use (eifu) are for illustrative purposes only and may differ slightly from the actual product you receive. While we strive to ensure the accuracy and completeness of the information provided, we cannot guarantee that the image accurately reflects the product at all times.

However, we want to assure you that through a continuous risk assessment process, we have evaluated that the potential deviations between the images and the actual product do not pose an increased risk when using the product. The instructions and essential information provided in the eifu remain valid and applicable, ensuring the safe and effective use of the product.

NORSE RESCUE®

Compressed Gauze Z-fold

 Anji Hengfeng Sanitary Material Co., Ltd
Ancheng, Dipu Town 313300 Anji County, Zhejiang,
China

EC **REP** Caretechion GmbH
Niederrheinstr. 71,
40474 Düsseldorf, Germany

REF HFEB01 (Z-fold)

 Do not use if the product sterile barrier system or its packaging is compromised

 Keep away from heat and sunlight

 Do not re-use

 No Latex in product

STERILE R Sterilized using irradiation.

 Indicates a single sterile barrier system

Instruction For Use

IFU VERSION:
CL-26_eIFU_20240424

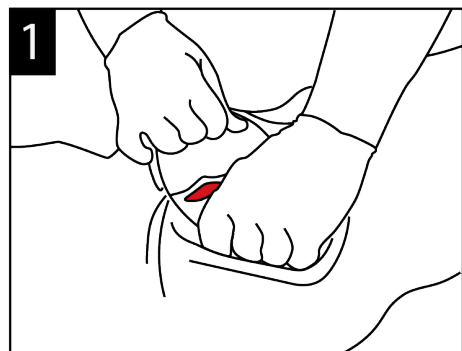
 English	 Norsk (Norwegian)	 Hrvatski (Croatian)
 Français (French)	 Suomi (Finnish)	 Eesti (Estonian)
 Español (Spanish)	 Ελληνικά (Greek)	 Latviešu (Latvian)
 Italiano (Italian)	 Български (Bulgarian)	 Lietuvių (Lithuanian)
 Português (Portuguese)	 Čeština (Czech)	 Slovenščina (Slovenian)
 Nederlands (Dutch)	 Slovenčina (Slovak)	 Malti (Maltese)
 Deutsch (German)	 Magyar (Hungarian)	 Gaeilge (Irish)
 Svenska (Swedish)	 Polski (Polish)	 Íslenska (Icelandic)
 Dansk (Danish)	 Română (Romanian)	 Україна (Ukraine)

 **English**


Instruction For Use

Open the Package: Locate the red tear points on the packaging. Tear at these points to open.

Prepare the Gauze: Remove the gauze from the package. Squeeze the gauze with one hand a few times to loosen the end and make it ready for use.

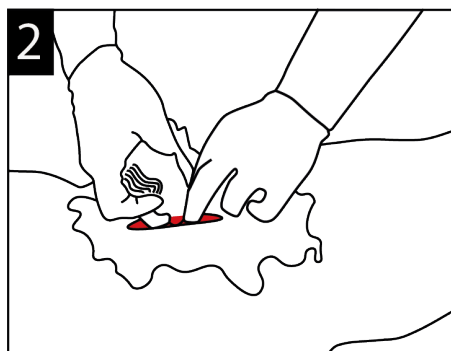Assess the Injury: Carefully examine the injury to ensure proper application of the gauze.

Apply the Gauze: Simultaneously pull out the necessary amount of gauze and compress it into the wound to help stop the bleeding.

Maintain Pressure: Continue to apply direct pressure on the wound site.

Secure the Gauze: Cover the gauze with a compression bandage. Monitor the wound regularly.

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