Electronic Instruction For Use

* Please note that the images in this online instruction for use (eifu) are for illustrative purposes only and may differ slightly from the actual product you receive. While we strive to ensure the accuracy and completeness of the information provided, we cannot guarantee that the image accurately reflects the product at all times.


However, we want to assure you that through a continuous risk assessment process, we have evaluated that the potential deviations between the images and the actual product do not pose an increased risk when using the product. The instructions and essential information provided in the eifu remain valid and applicable, ensuring the safe and effective use of the product.

NORSE RESCUE® EMT SHEARS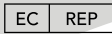

Symbols used (in compliance with ISO 15223-1)

-  Manufacturer (5.1.1)
-  Identification of medical device (5.7.7)
-  Catalogue No. (5.1.6)
-  Lot No. / Batch code (5.1.7)
-  UDI (Unique Device Identification) Carrier (5.7.10)
-  Manufacturing date (5.1.3) / Country of manufacture with country code defined in ISO 3166-1 (5.1.11)

-  Authorized representative in the European Community / European Union (5.1.2)
-  Distributor (5.1.9)
-  Importer (5.1.8)
-  Non-sterile (5.2.7)
-  CE Mark

Correct Disposal

Dispose of the trauma scissors in accordance with local regulations after their useful life or if they become damaged.

Instruction For Use

IFU VERSION:
EE-000331_eIFU_20240626

 GB - English 2	 NO - Norsk (Norwegian) 6	 HR - Hrvatski (Croatian) 11
 FR - Français (French) 2	 FI - Suomi (Finnish) 7	 EE - Eesti (Estonian) 11
 ES - Español (Spanish) 3	 GR - Ελληνικά (Greek) 7	 LV - Latviešu (Latvian) 12
 IT - Italiano (Italian) 3	 BG - Български (Bulgarian) 8	 LT - Lietuvių (Lithuanian) 12
 PT - Português (Portuguese) 4	 CZ - Čeština (Czech) 8	 SI - Slovenščina (Slovenian) 13
 NL - Nederlands (Dutch) 4	 SK - Slovenčina (Slovak) 9	 MT - Malti (Maltese) 13
 DE - Deutsch (German) . 5	 HU - Magyar (Hungarian) 9	 IE - Gaeilge (Irish) 14
 SE - Svenska (Swedish) 5	 PL - Polski (Polish) 10	 IS - Íslenska (Icelandic) 14
 DK - Dansk (Danish) 6	 RO - Română (Romanian) 10	 UA - Україна (Ukraine) 15

 **GB - English****Instruction For Use**

1. Product Description: Trauma scissors, featuring a plastic handle and a metal blade with a curved, blunted tip, are used by emergency personnel to cut tape, bandages, and other non-metallic materials. They are autoclavable at 134 degrees Celsius.

1.1 Intended Purpose: These scissors are for external use only in emergency situations, helping to quickly remove bandages or clothing without additional harm. They are not suitable for surgical or invasive procedures.

1.2 Limitations:

Not designed for surgical or invasive procedures. Avoid use on hard or metallic materials.

1.3  Warnings:

Improper use may result in injury.

1.4 Side Effects: Using trauma scissors as per instructions minimizes risk of side effects. Misuse may result in cuts or injuries.

1.5 Serious Incidents Must be Reported: All serious incidents related to the use of these scissors must be reported to the appropriate regulatory authorities and the manufacturer.

1.6 Storage: Store in a dry, cool place away from direct sunlight.

1.7 Disposal: Dispose of the trauma scissors in accordance with local regulations after their useful life or if they become damaged.

2. How to Use Instructions:

2.1 Insert your thumb and forefinger into the handle rings.

2.2 Spread handles to open, position around material, and close to cut.

2.3 Repeat as needed to complete cutting.

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