Electronic Instruction For Use

* Please note that the images in this online instruction for use (eifu) are for illustrative purposes only and may differ slightly from the actual product you receive. While we strive to ensure the accuracy and completeness of the information provided, we cannot guarantee that the image accurately reflects the product at all times.

However, we want to assure you that through a continuous risk assessment process, we have evaluated that the potential deviations between the images and the actual product do not pose an increased risk when using the product. The instructions and essential information provided in the eifu remain valid and applicable, ensuring the safe and effective use of the product.

NORSE RESCUE® Oropharyngeal Airway (OPA)

 WUXI EMSRUN TECHNOLOGY CO.,LTD
No.100 Fengbin Road,WUXI city,JIANGSU
province,China

 CR-OA03

 Do not re-use

 SUNGO Europe B.V.
Fascinatio Boulevard 522,
Unit 1.7, 2909VA Capelle aan den
IJssel, The Netherlands

Instruction For Use

IFU VERSION:
AW-00120_eIFU_20240918

 English	2	 Norsk (Norwegian)	6	 Hrvatski (Croatian)	11
 Français (French)	2	 Suomi (Finnish)	7	 Eesti (Estonian)	11
 Español (Spanish)	3	 Ελληνικά (Greek)	7	 Latviešu (Latvian)	12
 Italiano (Italian)	3	 Български (Bulgarian)	8	 Lietuvių (Lithuanian)	12
 Português (Portuguese)	4	 Čeština (Czech)	8	 Slovenščina (Slovenian)	13
 Nederlands (Dutch)	4	 Slovenčina (Slovak)	9	 Malti (Maltese)	13
 Deutsch (German)	5	 Magyar (Hungarian)	9	 Gaeilge (Irish)	14
 Svenska (Swedish)	5	 Polski (Polish)	10	 Íslenska (Icelandic)	14
 Dansk (Danish)	6	 Română (Romanian)	10	 Україна (Ukraine)	15

Instruction For Use

Product Description

The Oropharyngeal Airway is a disposable device designed to maintain upper airway patency. It is composed of polyethylene (PE) and comes in a PE outer package. It is indicated for patients with upper respiratory obstruction or unconsciousness following anesthesia.

Intended Purpose

The Oropharyngeal Airway is intended to keep the upper airway open in patients with airway obstruction or unconsciousness.

Patient Target Group

Suitable for patients with upper respiratory obstruction or unconsciousness, commonly after anesthesia.

Limitations

For single-use only and should not be reused.
Only for use by trained healthcare professionals.
Not suitable for use in toxic or hazardous environments.

Warnings

Select the correct size for the patient to avoid complications.
Avoid using in toxic or hazardous atmospheres.
Improper placement can cause airway obstruction or injury.

Precautions

Take care not to injure the teeth, lips, or tongue during insertion.

Contraindications

Not recommended for patients with intact airway reflexes, as it may cause gagging or vomiting.

Side Effects

Possible minor irritation to the mouth or throat.
Discomfort may occur in some patients.
Rare risk of oral injury if inserted improperly.

Serious Incidents Must Be Reported

Report any serious incidents to the manufacturer and relevant health authority according to local regulations.

Storage

Store in a cool, dry place. Avoid exposure to moisture and high temperatures. Shelf life: 5 years.

Disposal

Dispose of after single use in accordance with local regulations for medical waste.

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