Electronic Instruction For Use

* Please note that the images in this online instruction for use (eifu) are for illustrative purposes only and may differ slightly from the actual product you receive. While we strive to ensure the accuracy and completeness of the information provided, we cannot guarantee that the image accurately reflects the product at all times.

However, we want to assure you that through a continuous risk assessment process, we have evaluated that the potential deviations between the images and the actual product do not pose an increased risk when using the product. The instructions and essential information provided in the eifu remain valid and applicable, ensuring the safe and effective use of the product.

NORSE RESCUE®

Cohesive Bandage 7.5 cm x 4.5 m

 WUXI EMSRUN TECHNOLOGY CO., LTD
No.100 Fengbin Road, WUXI city
JIANGSU province, 214000, China

EC REP SUNGO Europe B.v.
Fascinatio Boulevard 522,
Unit 1.7, 2909VA Capelle aan
den IJssel, The Netherlands

REF CR-AB01-5

 Indicates that the product has not been subjected to a sterilization process.

 Keep away from heat and sunlight

 Do not re-use

 No Latex in product

Instruction For Use

IFU VERSION:
CL-001119_eIFU_20240903

 English	2	 Norsk (Norwegian)	3	 Hrvatski (Croatian)	5
 Français (French)	2	 Suomi (Finnish)	3	 Eesti (Estonian)	5
 Español (Spanish)	2	 Ελληνικά (Greek)	3	 Latviešu (Latvian)	5
 Italiano (Italian)	2	 Български (Bulgarian)	4	 Lietuvių (Lithuanian)	5
 Português (Portuguese)	2	 Čeština (Czech)	4	 Slovenščina (Slovenian)	5
 Nederlands (Dutch)	2	 Slovenčina (Slovak)	4	 Malti (Maltese)	5
 Deutsch (German)	3	 Magyar (Hungarian)	4	 Gaeilge (Irish)	6
 Svenska (Swedish)	3	 Polski (Polish)	4	 Íslenska (Icelandic)	6
 Dansk (Danish)	3	 Română (Romanian)	4	 Україна (Ukraine)	6

 **English****Instruction For Use**

Preparation: Ensure the skin around the area of application is clean and dry. If there is any broken skin, apply a protective dressing before using the bandage.

Application: Start wrapping the bandage from the farthest point from the heart, moving towards the center of the body. Overlap the bandage layers by at least 50% to ensure even coverage and secure adhesion. Ensure the bandage is snug but not too tight, allowing for normal circulation while providing adequate support and compression. The cohesive nature of the bandage will help it adhere to itself without needing additional fasteners.

Aftercare: Regularly check the extremity or area to ensure the bandage remains secure and has not become too tight. Reapply or adjust the bandage as needed to maintain proper support without restricting circulation.

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