Electronic Instruction For Use

* Please note that the images in this online instruction for use (eifu) are for illustrative purposes only and may differ slightly from the actual product you receive. While we strive to ensure the accuracy and completeness of the information provided, we cannot guarantee that the image accurately reflects the product at all times.

However, we want to assure you that through a continuous risk assessment process, we have evaluated that the potential deviations between the images and the actual product do not pose an increased risk when using the product. The instructions and essential information provided in the eifu remain valid and applicable, ensuring the safe and effective use of the product.

NORSE RESCUE®

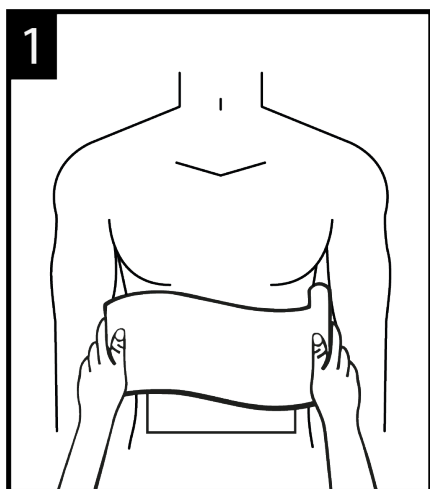
Trauma Bandage 8 Inches


 English	2	 Norsk (Norwegian)	11	 Hrvatski (Croatian)	20
 Français (French)	3	 Ελληνικά (Greek)	12	 Eesti (Estonian)	21
 Español (Spanish)	4	 Български (Bulgarian)	13	 Latviešu (Latvian)	22
 Italiano (Italian)	5	 Čeština (Czech)	14	 Lietuvių (Lithuanian)	23
 Português (Portuguese)	6	 Slovenčina (Slovak)	15	 Slovenščina (Slovenian)	24
 Nederlands (Dutch)	7	 Magyar (Hungarian)	16	 Malti (Maltese)	25
 Deutsch (German)	8	 Polski (Polish)	17	 Gaeilge (Irish)	26
 Svenska (Swedish)	9	 Română (Romanian)	18	 Íslenska (Icelandic)	27
 Dansk (Danish)	10	 Hrvatski (Croatian)	19		

Instruction For Use


ABDOMINAL

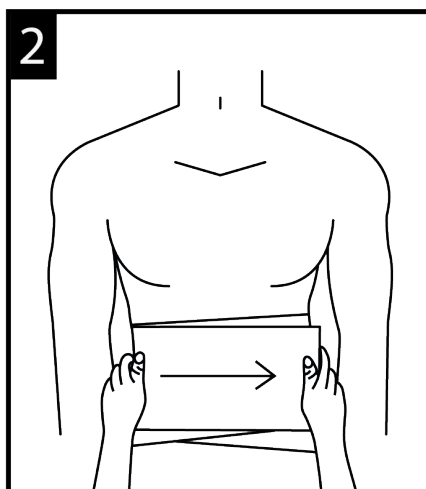1 Remove occlusive layer from pouch. Cover injury with occlusive layer. Then Place the 12 x 12 Inch pad on top of occlusive layer.

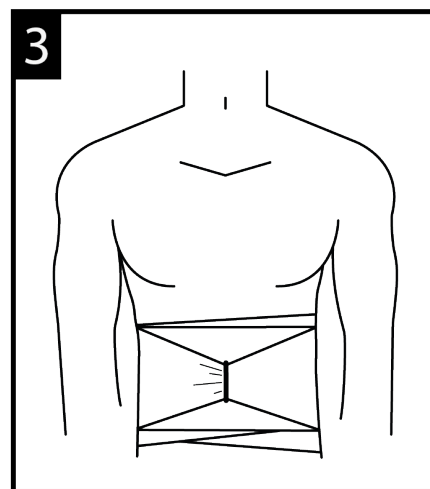
2 Secure with elastic bandage around the patient's abdominal area.


3 Close the bandage with the attached closure bar.

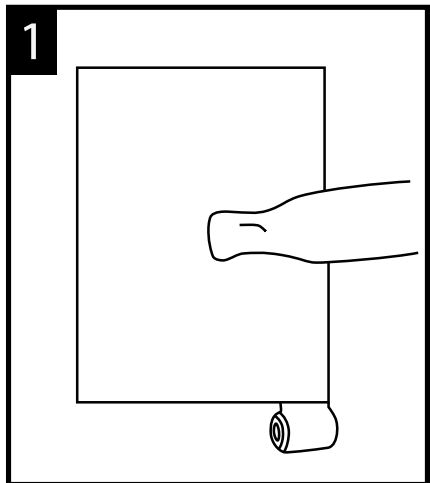
AMPUTATION

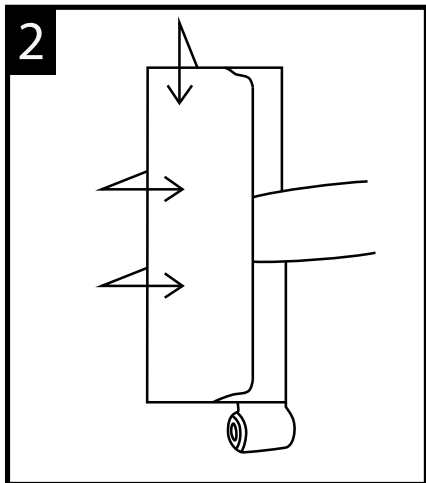1 Place the injured extremity in the center of the wound pad.

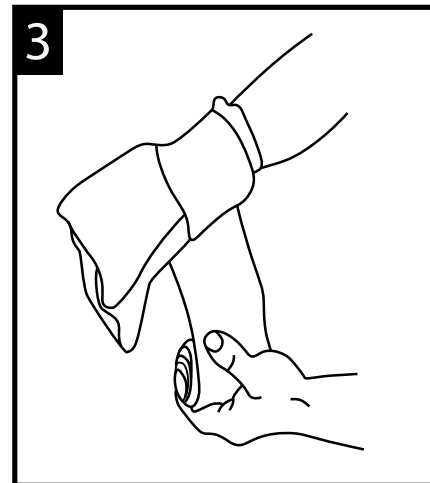
2 Fold the wound pad onto the extremity.

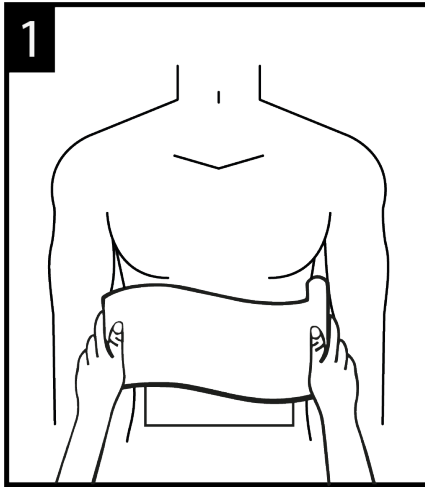3 Secure the wound pad with the elastic bandage.

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AP Services A/S